

BLEPHA CLEAN® PF

Sterile and 100% plant-based wipes*, soaked with micellar cleansing lotion.
For daily eyelid hygiene and sensitive skin.
Very good tolerance.

PACK CONTAINS

20 pre-impregnated sterile plant-based wipes* for single use only.

For cutaneous application.

* *Cellulose fibres*

PROPERTIES AND INDICATIONS

BLEPHACLEAN® PF is recommended for the daily hygiene of eyelids and sensitive skin and to keep eyelids clean before and after ophthalmic procedures.

BLEPHACLEAN® PF gently cleanses and removes dust, dirt, pollen and make-up, without irritating the eyes or damaging the outer layer of the skin.

BLEPHACLEAN® PF has a mix of ingredients to **soothe sensitive skin**.

BLEPHACLEAN® PF is for the daily hygiene of eyelids especially:

- when you have sensitive skin, eyelids or eyes.
- to respect the skin around the eyes and its natural bacterial flora.

Tolerance has been tested under ophthalmological, dermatological and paediatric control.

Fragrance-free and non-greasy, BLEPHACLEAN® PF is ready to use and does not need to be rinsed after use.

BLEPHACLEAN® PF can be used by contact lens wearers. BLEPHACLEAN® PF can be used by the entire family and on **babies over 3 months old**.

Instructions for use:

Use twice a day, morning and evening.

Use one wipe for each eye.

If you wear contact lenses, you must remove them before using BLEPHACLEAN® PF.

Wash hands thoroughly before using the wipes.

Tear across the sachet to open and unfold the wipe inside (Figure 1); do not use any tool to open the sachet.



Figure 1: Opening the sachet.

Wrap the wipe around the top of your index finger.

Keeping your eye closed, carefully place the wipe on your eyelids and the base of your eyelashes (Figure 2).

Gently massage with small circular movements, removing any crusts, secretions or make-up.



Figure 2: Cleansing your eyelids and eyelashes.

In front of a mirror, gently pull your lower eyelid down with your other hand and then rub your finger with the wipe wrapped around it along edge of your lower eyelid (Figure 3).

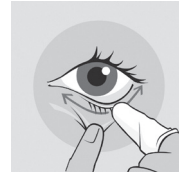


Figure 3: Cleansing the eyelid margin.

Repeat on the other eye, using a new wipe.

There is no need to rinse after using BLEPHACLEAN® PF wipe.

Precautions for use:

For cutaneous application.

Do not apply directly into the eyes.

Keep out of the sight and reach of children.

The colour of the wipe may vary. This is due to natural plant extracts and is normal.

Ingredients:

AQUA, PEG-8, POLOXAMER 184, POLYSORBATE 20, CAPRYLOYL GLYCINE, PROPYLENE GLYCOL, PEG-6 CAPRYLIC/CAPRIC GLYCERIDES, CENTELLA ASIATICA EXTRACT, IRIS FLORENTINA ROOT EXTRACT, SODIUM HYALURONATE, DIPOTASSIUM PHOSPHATE, POTASSIUM PHOSPHATE, ALCOHOL, ZINC SULFATE, SODIUM HYDROXIDE, HYDROCHLORIC ACID.

Manufacturer:

Laboratoires Théa

12, rue Louis Blériot

63100 Clermont-Ferrand

France

Tel. +33 (0)4 73 98 14 36

UK RP and marketed in UK

by:

Théa Pharmaceuticals Ltd

MED IC5 Innovation Way

Keele University Science &

Innovation Park

Keele - Staffordshire - ST5 5NT

UK

[www.thea-pharmaceuticals.](http://www.thea-pharmaceuticals.co.uk)

[co.uk](http://www.thea-pharmaceuticals.co.uk)

0345 521 1290

Made in France

©2023, Laboratoires Théa.

All rights reserved.

Théa®, BLEPHACLEAN® are
registered trademarks of
Laboratoires Théa.